
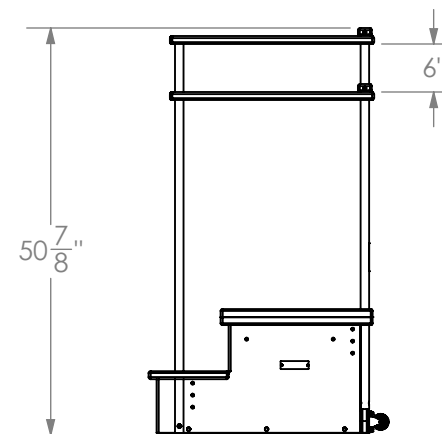
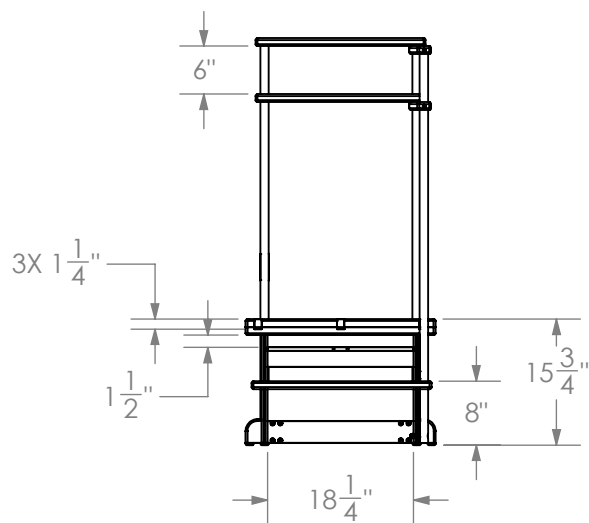
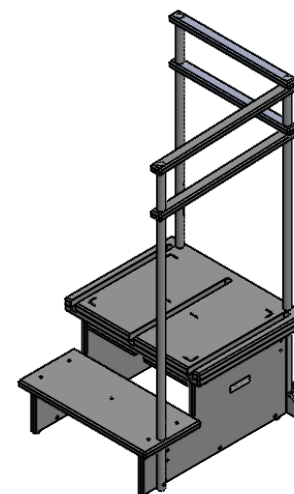


DETAIL A
SCALE 1 : 12

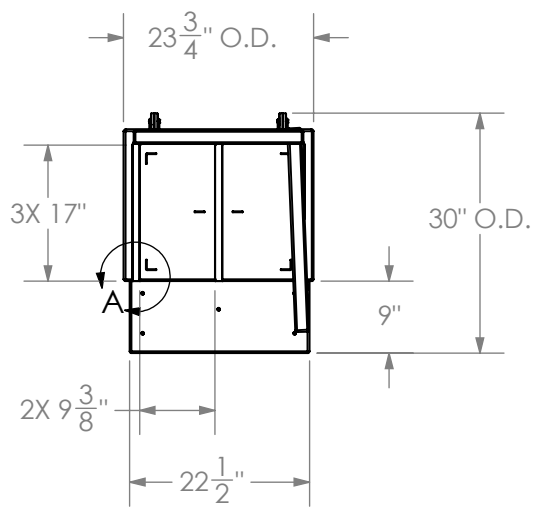

NOTE:

1. MATERIAL: POLYETHYLENE - LIGHT GREY
2. FOR USE WITH: COMBO STEP

THIRD ANGLE PROJECTION



UNLESS OTHERWISE SPECIFIED
DIMENSIONS ARE IN INCHES.

TOLERANCES ARE:
FRACTIONAL $\pm 1/16"$
ANGLES $\pm 1^\circ$
ONE PLACE DECIMAL $\pm .060$
TWO PLACE DECIMAL $\pm .030$
THREE PLACE DECIMAL $\pm .015$

DRAWN BY:

DATE:

CHECKED BY:

DATE:

APPROVED BY:

DATE:

MATERIAL:

SEE NOTES

BREAK ALL SHARP EDGES AND
REMOVE BURRS



CLEAR IMAGE DEVICES RESERVES
PROPRIETARY RIGHTS TO THIS DRAWING.
DRAWING AND THE DATA SHOWN THEREON
MAY NOT BE DISCLOSED, USED OR
REPRODUCED WITHOUT THE WRITTEN
CONSENT OF CLEAR IMAGE DEVICES.

TITLE:

MULTIVIEW POSITIONING PLATFORM

SIZE

A

DWG. NO.

24308-C50


REV

00

SCALE: 1:24

DO NOT SCALE DRAWING

SHEET 1 OF 2

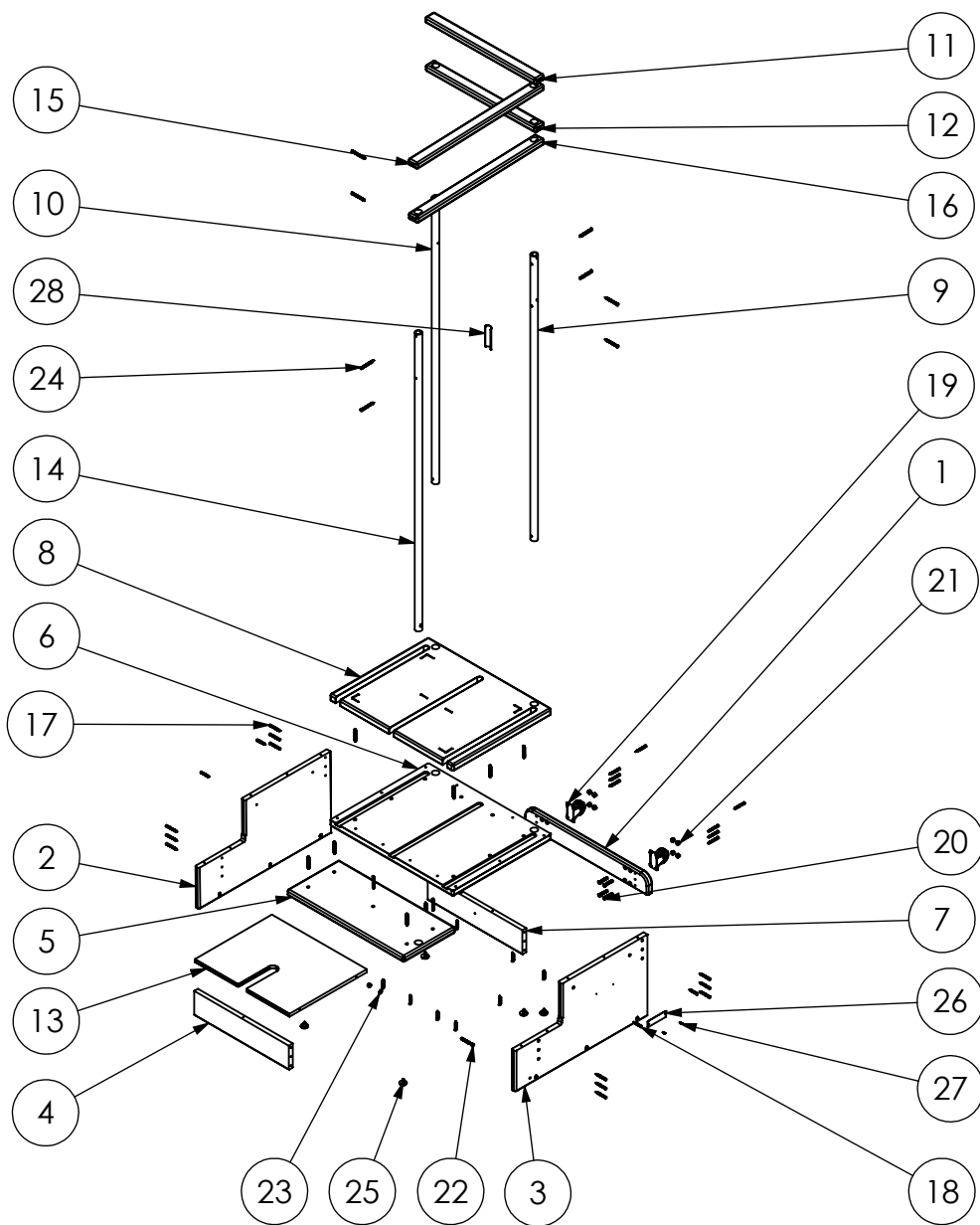

ITEM NO.	PART NO.	DESCRIPTION	24308-C50/QTY.
1	C-20304	24308 WHEEL BAR	1
2	C-20307	24308 LEFT SIDE	1
3	C-20308	24308 RIGHT SIDE	1
4	C-20310	24308 STEP BRACE	1
5	C-20303	24308 FIRST STEP	1
6	C-20300	24308-C50 PLATFORM BOTTOM	1
7	C-20309	24308 PLATFORM BRACE	1
8	C-20296	24308-C50 PLATFORM TOP	1
9	C-20312	3/4" X 47-1/2" SCH 40 REAR PIPE - GRAY	1
10	C-20074	3/4" X 47-1/2" SCH 40 STEP PIPE - GRAY	1
11	C-20305	24308 UPPER HANDLE	1
12	C-20306	24308 LOWER HANDLE	1
13	C-20294	DETECTOR SHELF	1
14	C-20311	3/4" X 49-1/2" SCH 40 SIDE RAIL PIPE - GRAY	1
15	C-20163	24601 TOP SIDE HANDLE	1
16	C-20148	24404 BOTTOM HANDLE	1
17	C-20015	#8 X 2" FH SQUARE DRIVE SCREW	33
18	C-20278	#8 X 1-1/2" FH SQUARE DRIVE SCREW	17
19	C-20089	2" RIGID CASTER	2
20	C-20260	1/4"-20 X 1-1/2" PHILLIPS FH SCREW	8
21	C-20090	1/4"-20 NYLON-INSERT LOCKNUT	9
22	C-20144	1/4"-20 X 2-14" PHILLIPS FH SCREW	1
23	C-20146	1/4" FLAT WASHER	1
24	C-20070	#10 x 2.5" FH SQUARE DRIVE SCREW	8
25	C-20073	1" RUBBER FEET WITH SCREW	6
26	C-20025	CLEAR IMAGE DEVICE LOGO PLATE	1
27	C-20016	#2 X .250" PH PHILLIPS SCREW	2
28	C-20314	24308-C50 PRODUCT LABEL	1

THIRD ANGLE PROJECTION



UNLESS OTHERWISE SPECIFIED
DIMENSIONS ARE IN INCHES.

TOLERANCES ARE:
FRACTIONAL $\pm 1/16"$
ANGLES $\pm 1^\circ$
ONE PLACE DECIMAL $\pm .060$
TWO PLACE DECIMAL $\pm .030$
THREE PLACE DECIMAL $\pm .015$

DRAWN BY:

DATE:

CHECKED BY:

DATE:

APPROVED BY:

DATE:

MATERIAL:

POLYETHYLENE

BREAK ALL SHARP EDGES AND
REMOVE BURRS



CLEAR IMAGE DEVICES RESERVES
PROPRIETARY RIGHTS TO THIS DRAWING.
DRAWING AND THE DATA SHOWN THEREON
MAY NOT BE DISCLOSED, USED OR
REPRODUCED WITHOUT THE WRITTEN
CONSENT OF CLEAR IMAGE DEVICES.

TITLE:

MULTIVIEW POSITIONING PLATFORM

SIZE

A

DWG. NO.

24308-C50

REV

00

SCALE: 1:26

DO NOT SCALE DRAWING

SHEET 2 OF 2