



MJM INTERNATIONAL CORPORATION

ASSEMBLY INSTRUCTIONS FOR

RECLINING SHOWER CHAIRS 193, 194, 195, 196

EXAMPLE - 196 COMPONENTS

IDENTIFY ALL COMPONENTS TO START ASSEMBLY



- 4 CASTERS W/ LEGS STEMS
(2 BRAKE & 2 NON-BRAKE)
- 1 SLIDING FOOTREST SUPPORT
- 2 ARMREST
- ELEVATED LEGREST /WITH CUSHION.
- RECLINING BACKREST /LUMBAR SUPPORT
CUSHION /SLING SUPPORT.
- PUSH PULL HANDLE
- 16 (SIXTEEN) 1/2" SCREWS.
- 4 (FOUR) 1" SCREWS.

STEP 2



- PLACE 196 RECLINING SHOWER CHAIR ON A FLAT SURFACE UPSIDE DOWN TO INSERT (4) CASTERS.
- SECURE CASTERS USING 1/2" RETAINING SCREWS.
(USE PREDRILLED HOLES AS REFERENCE).

NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN / SECURE ALL SCREWS.

NOTE: PLEASE MATCH EACH COLOR-CODED DOT ACCORDINGLY.

STEP 3



- REMOVE 1/2" RETAINING SCREWS, (4) FROM 4-WAY FITTINGS TO INSTALL SLIDING FOOTREST.
- CENTER AND INSERT SLIDING FOOTREST.
- SECURE SLIDING FOOTREST USING 1/2" RETAINING SCREWS.
(USE PREDRILLED HOLES AS REFERENCE).

STEP 4



- REMOVE (2) 1" RETAINING SCREWS FROM ELEVATED LEGREST SUPPORT AND SEPARATE FROM PVC PIPE.
- CENTER AND INSERT LEGREST SUPPORT INTO SWIVEL FITTINGS.
- SECURE ELEVATED LEGREST SUPPORT USING 1" RETAINING SCREWS.
(USE PREDRILLED HOLES AS REFERENCE).



196 RECLINING SHOWER CHAIR
FACILITATE SHOWERING & TOILETING
SHOWN WITH
OPTIONAL COMMODORE PAIL



MJM INTERNATIONAL CORPORATION

ASSEMBLY INSTRUCTIONS FOR

RECLINING SHOWER CHAIRS 193, 194, 195, 196

STEP 5



- INSERT ARMREST INTO SEAT SECTION AND SECURE USING 1/2" RETAINING SCREWS. (USE PREDRILLED HOLES AS REFERENCE).

STEP 6



- INSERT RECLINING BACKREST INTO SWIVEL FITTINGS AND SECURE USING 1/2" RETAINING SCREWS. (USE PREDRILLED HOLES AS REFERENCE).

STEP 7



- INSERT PUSH PULL HANDLE INTO BACK SECTION AND SECURE USING 1/2" RETAINING SCREWS. (USE PREDRILLED HOLES AS REFERENCE).



194 RECLINING SHOWER CHAIR

STEP 8



- PLACE CUSHION ON ELEVATED LEGREST SUPPORT AND SECURE VELCRO STRAPS.
- PLACE LUMBAR SUPPORT ON RECLINING BACKREST AND SECURE WITH SNAPS.
- PLACE SLING SUPPORT ON RECLINING BACKREST.

NOTE: CASTERS, RECLINING BACKREST, ELEVATING LEGREST, SLIDING FOOTREST & PUSH PULL HANDLE MUST BE CHECKED AND SECURED PRIOR TO USING SHOWER CHAIR.

WARNING NOT SECURING ALL THE ABOVE CAN CREATE A SERIOUS SAFETY HAZARD.