

HOW TO INSTALL
RADIATION SHIELDING PLEATED TABLE SHIELD

ATTENTION

BEFORE INSTALL THE RADIATION SHIELDING PLEATED TABLE SHIELD, YOUR IMAGING TABLE MUST BE CLEAN AND DRY.

WE RECOMMEND CLEANING YOUR IMAGING TABLE WITH CAVI WIPES.

1

WITH THE NON-SLIP PAD
FACING THE IMAGING TABLE.
PEEL THE PROTECTIVE FILM




STEPS

HOW TO INSTALL
RADIATION SHIELDING PLEATED TABLE SHIELD

STEPS

2

**POSITION THE LEAD CURTAIN
ALIGNED WITH THE CENTER OF
YOUR TABLE.**

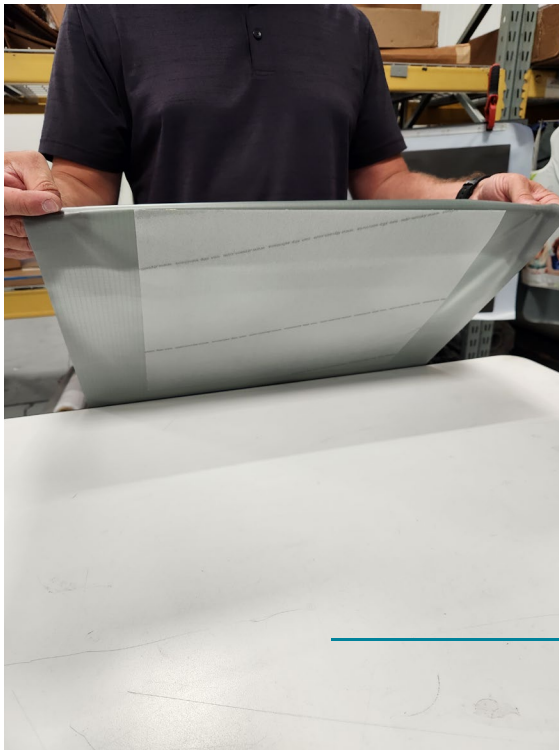

REMEMBER
The table surface
MUST BE completely
clean and dry.

3

**ATTACH THE ENTIRE SURFACE
WITH THE TABLE.**



HOW TO INSTALL
RADIATION SHIELDING PLEATED TABLE SHIELD

STEPS

4

**PRESS FIRMLY TO ENSURE
ADHERANCE.**

