

# INSTRUCTIONS FOR USE

## SIX SIMPLE STEPS TO GUARANTEED SPEED, ACCURACY, AND PRECISION

**STEP ONE:** Remove backing from tape.

**STEP TWO:** Place and scan over the area of interest.

**STEP THREE:** Notice the twelve dots on the skin surface.

**STEP FOUR:** Determine which dot best represents the point of entry.

**STEP FIVE:** Mark through the grid with felt tip pen where laser crosses the line.

**STEP SIX:** Remove the grid and perform your biopsy.

