



RXWarmth Blanket Warmer - Usage and Care Instructions

* The use of Non-OEM components is strictly prohibited and will void all warranties, both Express and Implied. Electric RXWarmth Blanket Warmers should only be used with the HBPS-1 (AC Pwr. Supply), and/or HBPC-AMP-72 (12vDC Pwr. Cord) *

GENERAL INSTRUCTIONS

- 1.) Place blanket(s) into the RXWarmth bag. Close front flap securely by pressing Velcro strips together.
- 2.) Insert 12vDC power cord into your vehicles lighter outlet, or plug the purchased AC power supply into an available AC outlet.
- 3.) Attach power cord to the RXWarmth bag using the white connector located on the front of the bag.
- 4.) If using a 12vDC plug, make sure the plug adapter fits securely into the lighter outlet (a loose connection could damage plug and stop power to the Blanket Warmer).

WASHING INSTRUCTIONS

- 1.) Remove Heating Elements using access panel found on the rear of the bag (zippered opening).
- 2.) RXWarmth bags can be washed using a commercial washing machine, such as those found at your local
- 3.) Laundromat. Pre-treat any heavily soiled areas with a dish detergent such as Dawn. This will help loosen existing grease and dirt, In turn making the machine wash more effective.
- 4.) Wash bags on "Normal" cycle, using HOT water and a standard detergent.
- 5.) After bags have gone through a complete cycle, check the bags for badly soiled areas which may not have come clean with a single treatment, and wash again.

DRYING INSTRUCTIONS

- 1.) Make sure the front flap of the HOTBAG is closed, and that the rear access pouch is unzipped and left open.
- 2.) Dry bags on MED-HIGH heat for approx. 20 minutes.
- 3.) If bags are still damp, install the heating elements in the bag and plug the bag into a power supply. This will help dry the bags from the inside out, and will reduce the risk of mildew.

DO NOT

- 1.) Do Not stand bags on end
- 2.) Do Not set any items on top of the bag.
- 3.) Do Not use dishwasher or dryer.
- 4.) Do Not dry clean or steam clean.
- 5.) Do Not place sharp objects in or on the RXWarmth bag.
- 6.) Do Not leave RXWarmth bag plugged into 12v outlet when not in use (may discharge battery).

TROUBLE SHOOTING

- 1.) **12vDC Power Cord (HBPC-AMP-72):** Turn 12vDC plug inside the lighter socket to improve connection. Green LED will illuminate on the power plug when a proper connection is made.
- 2.) Check fuse located in the cigarette plug (unscrew tip - turn counter clockwise to remove fuse)
- 3.) Check fuse of your vehicles cigarette lighter, located in the main fuse panel.
- 4.) **Desktop Power Supply (HBPS-1):** Check connection to your power supply and wall outlet (applies only when using the AC Power Supply).
- 5.) Check power cord and plug for broken or frayed wires.
- 6.) Make sure connector contacts are clean and free of corrosion.
- 7.) If your RXWarmth blanket warmer does not heat up after reviewing the above steps, contact Z&Z Medical, Inc. at 1.800.410.9575.

IEC 60601-1 Medical Accessory Compliant