


AGR-105 / AGR-110 / AGR-115

ASSEMBLY INSTRUCTIONS

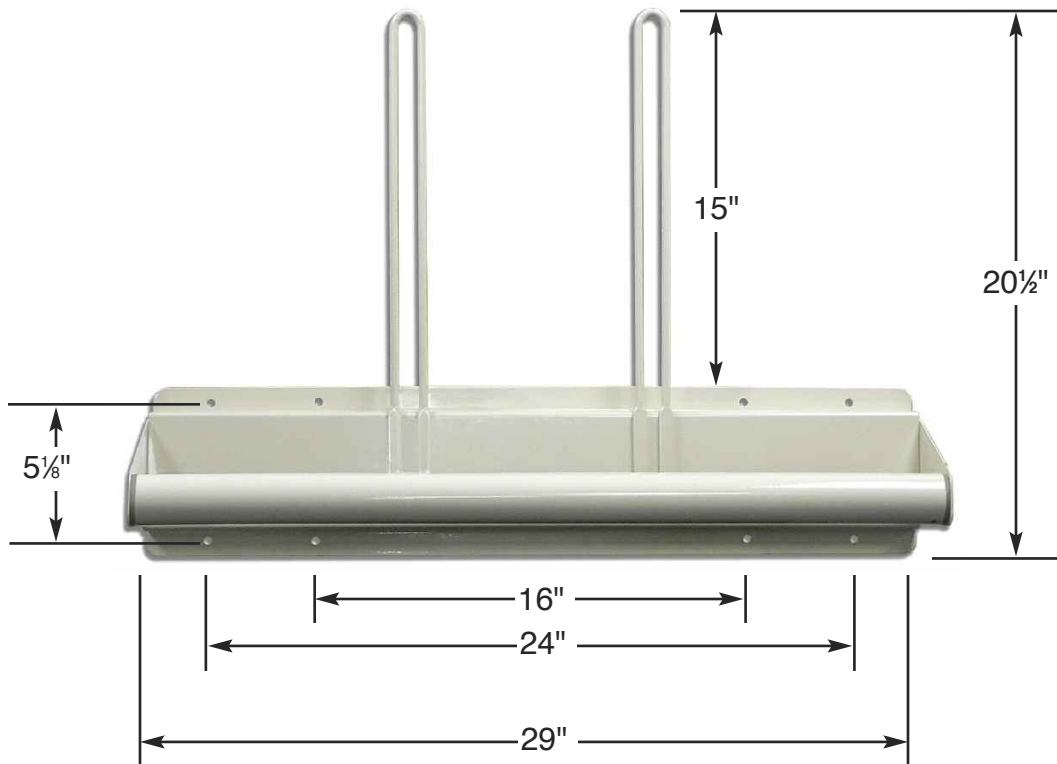

(1) Install the glove holders onto the back plate as shown above. The AGR-110 uses two glove holders and the AGR-115 uses four glove holders with the head of the screws facing outward and the nuts on the reverse side. The AGR-105 does not have glove holders.

(2) It is **IMPORTANT** that you find the **STRUCTURAL STUDS** in the wall before mounting this unit.

Most walls are built with studs on either 16" or 24" centers. Therefore, you must use the 16" or 24" holes to **SECURELY** mount this unit to your wall.

The excessive weight of lead aprons demands that you follow this procedure to insure proper function and utilization of the unit and also preserve the quality of your wall.

