



We, at the Harloff Company, strive to provide the highest quality products and instruction available. Below you will find a list of standard cleaning solutions that may be used on our line of powder coat painted and stainless steel carts and cabinets, as indicated below. Glass surfaces and the exterior of the cart can be cleaned with these items but is not suitable for autoclave or other high temperature machines.

Mild detergent and water
Alcohol wipes
Alcohol solutions
Lysol Wipes
Super Sani-Cloth Germicidal Wipes
Sani-Cloth Plus Wipes
Virex II
Glance
Windex
Oxivir Five
Hydrogen Peroxide (3.0%)

For Scope Transport Trays, including STC-SC249G (Gray Trays), STC-SC-249L (Gray Lids) and STC-SC152L (Clear Lids) : Mild detergent & water, Soap & water

For acrylic/polycarbonate surfaces used in clear drawer faces, clear bins and clear tray covers: Soap and water Only

For wood vinyl surfaces: Non-abrasive household cleaners, Formula 409® (The Clorox Co.), Fantastik® (S.C. Johnson) and Simple Green® (Sunshine Makers).

DO NOT USE bleach or any bleach-based products to clean Harloff carts and cabinets.

Please feel free to contact us if you have any further needs or require information on a cleaner not listed above. We will be happy to look into a product with a copy of the MSDS sheet you provide.

Regards,

The Harloff Team