




"We Do Care - We Do More"

MJM International Corporation

AW-S-M-L

Assembly Instructions

IDENTIFY COMPONENTS TO START ASSEMBLY

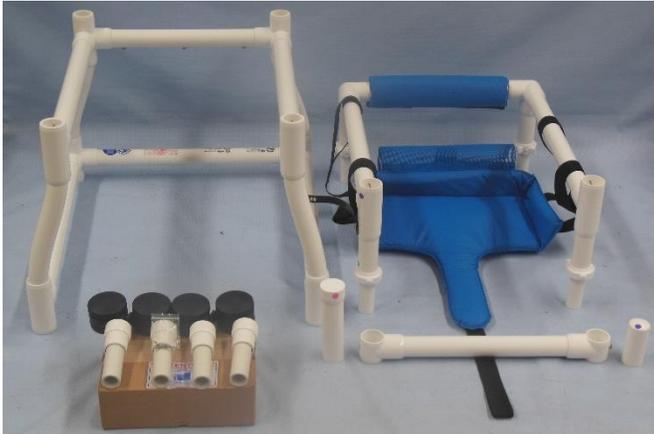

- **BOTTOM FRAME.**
- **TOP FRAME WITH SEAT.**
- **FRONT GATE WITH PVC POST.**
- **BOX WITH CASTERS AND SCREWS**
 - 2 CASTERS WITH BRAKES.
 - 2 CASTERS NON BRAKE.

STEP 1 – CASTERS



- **INSERT CASTER SOCKETS INTO BOTTOM FRAME.**

NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN / SECURE ALL SCREWS.

STEP 2 CASTERS.




- **DRILL THROUGH CASTER SOCKETS USING 1/8 DRILL BIT AND PILOT HOLES AS REFERENCE.**



- **SECURE USING 1/2" RETAINING SCREWS AND PILOT HOLES AS REFERENCE.**

STEP 3 STROLLER TOP FRAME ASSEMBLY

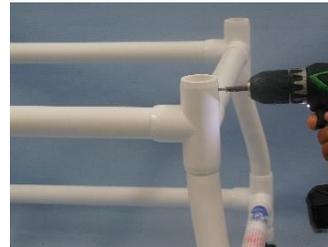

- **REMOVE SCREWS FROM BOTTOM FRAME.**
- **INSERT TOP FRAME INTO BOTTOM FRAME, ALIGN PREDRILLED HOLES.**
- **SECURE WITH SCREWS PROVIDED.**





MJM International Corporation

AW-S-M-L

"We Do Care - We Do More"

Assembly Instructions


STEP 4 SEAT & GATE



- SECURE SEAT WITH SNAPS PROVIDED.
- REMOVE SCREW FROM CORNER T FITTING.
- INSERT PVC PIPE INTO CORNER T FITTING, ALIGN USING COLOR CODED DOTS.



STEP 5 GATE.

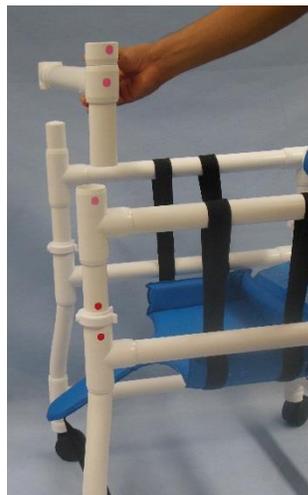

- INSERT PVC PIPE INTO CORNER T FITTING, ALIGN USING COLOR CODED DOTS.
- SECURE PVC PIPE WITH SCREW PROVIDED.

STEP 6 SEAT STRAP



- SECURE SEAT STRAP BY SLIDING IT OVER THE FRONT GATE BAR.



- SECURE GATE BAR WITH STRAP PROVIDED.

STEP 7



- PRODUCT COMPLETELY ASSEMBLED

NOTE: CASTERS, TOP FRAME, BOTTOM FRAME, SEAT, SEAT STRAP, GATE, MUST BE CHECKED AND SECURED PRIOR TO USING SHOWER CHAIR.

Rev: 1.3

323 East Owassa • Suite 4 • Edinburg, TX 78539 • 1-800-270-6199 • 956-781-5000 • Fax 956-781-5552

www.mjminternational.com • hmoroles@yahoo.com