



# MJM INTERNATIONAL CORPORATION

## 418-OR-3TW

**WARNING NOT SECURING ALL PARTS PROPERLY CAN CREATE A SERIOUS SAFETY HAZARD.**

**IDENTIFY ALL COMPONENTS TO START ASSEMBLY**  
**PARTS INCLUDED FOR WALKER**

- PARTIALLY ASSEMBLED FRAME.
- 4 CASTERS (2 WITH BRAKE & 2 NON BRAKE
- SIDE OUTRIGGERS WITH ANTI-TIPS--2
- SIDE RAILS--- 2 MAKE UP THE ARM SECTIONS
- SPRING LOADED FLAT GATE---1 IS CONNECTED TO ONE SIDE RAIL
- PVC BACK --1
- MESH SLING-- 1
- SEAT CUSHION--1
- SAFETY BELT-- 1
- CLAMP LOCKS FOR ADJUSTABLE HEIGHTS--- 4 PLACED ON BOTTOM FRAME
- REINFORCEMENT RINGS -- 4 ATTACHED TO BOTTOM FRAME IN CORNERS

STEP 1



**NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN/SECURE ALL SCREWS.**

STEP 2



**REMOVE SCREWS FROM REINFORCEMENT RINGS**

STEP 3



**INSERT CASTERS --PLACE BRAKE WHEELS ON THE REAR OF THE WALKER**

STEP 4



**SECURE CASTERS WITH SCREWS PROVIDED USING PILOT HOLE AS REFERENCE**

STEP 5



**INSERT OUTRIGGERS ON THE SIDE OF THE BOTTOM FRAME BY MATCHING COLOR CODED DOTS AND SECURE USING SCREWS PROVIDED USING PILOT HOLES AS REFERENCE**

STEP 6



**REMOVE SCREWS FROM AREA BY SEAT SECTION**

STEP 7-1



**INSERT SIDE RAILS BY MATCHING COLOR CODED DOTS.**



# MJM INTERNATIONAL CORPORATION

## 418-OR-3TW

**WARNING NOT SECURING ALL PARTS PROPERLY CAN CREATE A SERIOUS SAFETY HAZARD.**

STEP 7-2



INSERT SIDE RAILS BY MATCHING COLOR CODED DOTS .

STEP 7-3



SECURE SIDE RAILS USING SCREWS PROVIDED AND PILOT HOLES AS REFERENCE

STEP 8



INSERT MESH SLING WITH PVC BACK TOWARDS SEAT SECTION ON TOP OF SIDE RAILS, SECURE USING SCREWS PROVIDED AND PILOT HOLES AS REFERENCE.

STEP 9



ADJUST MESH SLING AS NEEDED

STEP 10



SECURE FLAT GATE ON FRONT SIDE RAIL USING PILOT HOLE PROVIDED

STEP 11



418-OR-3TW COMPLETELY ASSEMBLED

**NOTE: CLAMP LOCKS CAN BE REMOVED TO ADJUST THE HEIGHTS ON THE WALKER.  
NOTE: SECURE SAFETY BELT ON THE FRONT GATE AND SECURE SEAT CUSHION WITH VELCRO STRAPS**

**NOTE: CASTERS/CLAMP LOCKING SYSTEM MUST BE CHECKED AND SECURED PRIOR TO USING 418-OR-3TW**

MJM International Corporation  
[www.mjminternational.com](http://www.mjminternational.com)

1-800-270-6199

Fax: (956) 781-5552  
[hmoroles@yahoo.com](mailto:hmoroles@yahoo.com)