



MJM INTERNATIONAL CORPORATION

ASSEMBLY INSTRUCTIONS FOR

418-3, 421-3 WALKERS

IDENTIFY ALL COMPONENTS TO START ASSEMBLY



- ☐ 4 CASTERS W/ LEGS STEMS (2 BRAKE & 2 NON-BRAKES)
- ☐ 1 SEAT SECTION
- ☐ 2 ARMREST
- ☐ 1 BACKREST
- ☐ 1 LAP SECURITY BAR
- ☐ 1 MESH SLING & CUSHION
- ☐ 1 SAFETY SECURITY STRAP
- ☐ 4 (FOUR) 1/2" SCREWS.
- ☐ 8 (EIGHT) 1" SCREWS.

NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN/SECURE ALL SCREWS.

*** PLEASE MATCH EACH COLOR-CODED DOT ACCORDINGLY.**

STEP 2



- ☐ INSERT CASTER LEG STEMS INTO BASE.
- ☐ DRILL CASTER LEG STEMS USING 1/8" DRILL BIT (USE PREDRILLED HOLES AS REFERENCE).
- ☐ SECURE CASTERS WITH 1" RETAINING SCREWS.

STEP 3



- ☐ INSERT ARMREST INTO SEAT SECTION.
- ☐ DRILL PVC PIPE ADJUSTMENT (USE PREDRILLED HOLES AS REFERENCE) & SECURE USING 1" RETAINING SCREWS.

STEP 4



- ☐ INSERT SMALL PVC PIPE INTO T-FITTING ON RIGHT FRONT SIDE OF ARMREST.
- ☐ DRILL (USING PREDRILLED HOLES AS REFERENCE) & SECURE USING 1" RETAINING SCREWS.
- ☐ INSERT ENTRANCE GATE BAR ON SMALL PVC PIPE.

STEP 5



- ☐ INSERT LARGE PVC PIPE THROUGH SLIP-T FITTING OF ENTRANCE GATE BAR & INTO T- FITTING ON LEFT FRONT SIDE OF ARMREST.
- ☐ DRILL (USING PREDRILLED HOLES AS REFERENCE) & SECURE USING 1" RETAINING SCREWS.

STEP 6



- ☐ INSERT BACKREST INTO ARMREST SECTION.
- ☐ DRILL PVC BACKREST (USE PREDRILLED HOLES AS REFERENCE) SECURE BACKREST USING 1/2" RETAINING SCREWS.

STEP 7



- ☐ PLACE CUSHION SEAT; SECURE STRAP & CLIPS ON BACK.
- ☐ INSERT STRAP THROUGH ENTRANCE GATE BAR.



- ☐ PLACE MESH SLING ON PVC BACKREST & SECURE STRAP ON BACK.

418-3 COMPLETELY ASSEMBLED

NOTE: CASTERS, ARMREST & WIRE LOCK PIN MUST BE CHECKED AND SECURED PRIOR TO USING 418-3, 421-3 WALKER.

WARNING NOT SECURING ALL THE ABOVE CAN CREATE A SERIOUS SAFETY HAZARD.

MJM International Corporation

1-800-270-6199

Fax: (956) 781-5552

www.mjminternational.com

REV. B

Date: 04-14-09

Page 1 of 1