

# 450-PED, CHILD AND ADULT PLATFORM WALKER ASSEMBLY INSTRUCTIONS

**WARNING!!!!!! CASTERS, REMOVABLE VELCRO PADDED TRAY, TOP AND BOTTOM SECTIONS MUST BE CHECKED AND SECURED PRIOR TO USING 450-PED, CHILD AND ADULT PLATFORM WALKER.**

## IDENTIFY COMPONENTS TO START ASSEMBLY

### PARTS INCLUDED

- ☐ 1 REMOVABLE VELCRO PADDED TRAY.
- ☐ 5 CASTERS WITH LEG STEMS (IN BROWN BOX)  
(2 BRAKE & 3 NON-BRAKE).
- ☐ 1 BOTTOM SECTION.
- ☐ 1 TOP SECTION.
- ☐ 5 REINFORCEMENT RINGS.
- ☐ 4 – 6 x 3/8" RETAINING SCREWS (ON FRAME).
- ☐ 10 – 8 x 1" RETAINING SCREWS (IN BROWN BOX).

## STEP 1 – IMAGE REFERENCE



**NOTE: A PHILLIPS SCREWDRIVER OR A CORDLESS HAND DRILL IS REQUIRED TO TIGHTEN/SECURE ALL SCREWS.**

## STEP 2 – ASSEMBLING CASTERS.



- ☐ REMOVE 8 x 1" SCREWS FROM REINFORCEMENT RINGS,  
**NOTE: PLEASE MATCH EACH COLOR –CODED DOTS ACCORDINGLY.**
- ☐ INSERT REINFORCEMENT RINGS INTO CASTER SECTION ON BOTTOM FRAME (ALIGN RING WITH PREDRILLED HOLES).
- ☐ INSERT CASTER LEG STEMS INTO BASE SECTION.
- ☐ SECURE THE RING AND CASTERS WITH THE 8 x 1" RETAINING SCREWS.

## STEP 3 – ASSEMBLING TOP AND BOTTOM SECTION.



- ☐ REMOVE FOUR 6 x 3/8" SCREWS FROM MIDDLE SECTION.
- ☐ INSERT TOP SECTION INTO BOTTOM FRAME.
- ☐ SECURE TOP SECTION USING THE FOUR 6 x 3/8" SCREWS.

# 450-PED, CHILD AND ADULT PLATFORM WALKER ASSEMBLY INSTRUCTIONS

## STEP 4 – ASSEMBLING REMOVABLE VELCRO PADDED TRAY.



- UNDO VELCRO ON PADDED TRAY.
- SECURE VELCRO ON PADDED TRAY.

## STEP 5 – ADJUSTING HAND GRIPS.



- REMOVE WIRE LOCKING PINS FROM FRAME TO ROTATE HAND GRIPS AND TO ADJUST WIDTH.
- RESECURE USING WIRE LOCKIN PIN.



450-ADULT PLATFORM WALKER COMPLETELY ASSEMBLED.



**NOTE: TO ADJUST 450-PED, CHILD AND ADULT PLATFORM WALKER PLEASE REMOVE THE FOUR CLAMP LOCKING SYSTEMS AND ADJUST ACCORDINGLY.**

**NOTE: CASTERS LEG STEMS, REMOVABLE VELCRO PADDED TABLE, TOP AND BOTTOM SECTION MUST BE CHECKED AND PROPERLY SECURED PRIOR TO USING 450-PED, CHILD AND ADULT PLATFORM WALKER.**

*Thank you for purchasing MJM International products; we appreciate the opportunity to serve your needs. To view our entire product lines please visit our website at [www.miminternational.com](http://www.miminternational.com)*