



USE & CARE OF BAR-RAY PROTECTIVE APPAREL

Thank you for choosing our products. To ensure the longevity of your new garment, please follow these simple guidelines:

- Radiograph your garment upon receipt and regularly after that date to insure no damage has occurred to the protective core.
- Hang your apron properly either by the shoulders or by straps depending on the style.
- Do not fold or crease the garment.
- Hook & loop fasteners should be matched together so they do not snag or tear the fabric.
- Do not expose the garment to extreme temperatures or submerge in liquids.
- Do not lean or rub against sharp corners/edges or store sharp objects in the pockets.

WARRANTY

Bar-Ray guarantees its products against defects in materials and workmanship for a period of two (2) years starting from date of factory shipment.

DISPOSAL

We offer disposal of your old garments on a one-for-one basis (buy one/return one). Remove the outer cover, front & back, keeping the manufacturing tag intact and return prepaid to the address below. Written preauthorization is required or shipments will be refused.

QUESTIONS

Please contact us.

ISO 9001

- This product was manufactured in a plant whose quality management system is registered to ISO 9001:2008

CE

- Article 10/11 Notified Body: 0338, BTTG, Unit 14, Wheel Forge Way, Manchester, M171EH, United Kingdom
- Identifies this product has been certified according to the PPR Directive, 89/686/EEC
- Signifies that this product offers protection against medical x-rays and that the protective material used is at a lead equivalence as shown on the garment label.
- Materials used in production of our products have been tested using methods equivalent to EN 61331-1:2002