



Techno-Aide, Inc.  
7117 Centennial Boulevard  
Nashville, TN 37209  
P: 800.251.2629  
F: 800.554.6275

---

## **SandBag & SandSock CLEANING & CARE INSTRUCTIONS 2015**

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new sandbags and sandsocks, we recommend the following guidelines for cleaning and care.

### **Cleaning & Care:**

We all know that in a medical environment, it is easy for vinyl's to become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

#### **Do:**

- Techno-Aide recommends cleaning with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean the surface fabric or vinyl and remove dust and dirt from your table pad.
- To disinfect your sandbags and sandsocks, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth AF3 Alcohol Free Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes Store your sandbags and sandsocks on a sandbag/sandsock rack when not in use.

#### **Don't:**

- Do not use Bleach or any harsh chemicals on your sandbags and sandsocks.
- Do not autoclave or launder.
- Do not soak/submerge table pad in water.

Failure to adhere to the above recommendations will shorten the life of your sandbags and sandsocks and will possibly void your warranty. Techno-Aide provides a one year warranty on workmanship and materials.