

## **Bolster Sponges**

### **CLEANING & CARE INSTRUCTIONS**

#### **2015**

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new bolster sponges, we recommend following these guidelines for cleaning and care.

#### Cleaning & Care:

We all know that in a medical environment, it is easy for surfaces to become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

#### **Do:**

- Techno-Aide recommends cleaning with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean and remove dust and dirt from your coated sponge.
- To disinfect your sponges, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth AF3 Alcohol Free Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes.
- PDI Sani-Cloth Bleach wipes can be used as a disinfectant if you strictly follow the manufactures guidelines. Failure to comply with these guidelines will result in void of warranty.
- Store your sponges in a clean and dry area when not in use.

#### **Don't:**

- Do not autoclave or launder.
- Do not soak/submerge sponges in water.

Failure to adhere to the above recommendations will shorten the life of your bolster sponges, promote discoloration of the surface, and will possibly void your warranty. Techno-Aide provides a one year warranty on workmanship and materials.