



Techno-Aide, Inc.
7117 Centennial Boulevard
Nashville, TN 37209
P: 800.251.2629
F: 800.554.6275

Stealth Sponges

CLEANING & CARE INSTRUCTIONS

2016

Thank you for choosing Techno-Aide for your radiology accessory needs. To ensure the longevity of your new Stealth-Cote sponges, we recommend following these guidelines for cleaning and care.

Cleaning & Care for Stealth Cote Sponges :

We all know that in a medical environment, it is easy for surfaces to become dirty and soiled from blood, bodily fluids, betadine, barium, and other fluids.

Do:

- Techno-Aide recommends cleaning with cold water and a mild detergent immediately after contact with fluids. Household 409 Spray can be used to clean and remove dust and dirt from your coated sponge.
- To disinfect your sponges, Techno-Aide recommends the use of disinfectant wipes such as PDI Sani-Cloth AF3 Alcohol Free Germicidal Wipes Cat# GWP-WAF. Allow a drying time of at least 3 minutes.
- PDI Sani-Cloth Bleach wipes can be used as a disinfectant if you strictly follow the manufactures guidelines. Failure to comply with these guidelines will result in void of warranty.
- Store your sponges in a clean and dry area when not in use.

Don't:

- Do not autoclave or launder.
- Do not soak/submerge sponges in water.

Cleaning & Care for Stealth Core Sponges :

These sponges do not have a surface that is classified as "cleanable". You can however spot clean the foam fairly well with Formula 409 Cleaner/disinfectant without affecting the foam in any negative ways.

Failure to adhere to the above recommendations will shorten the life of your sponges, promote discoloration of the surface, and will possibly void your warranty. Techno-Aide provides a one year warranty on workmanship and materials.

