


ARW-10 / ARW-L5 / ARW-R5

ASSEMBLY INSTRUCTIONS

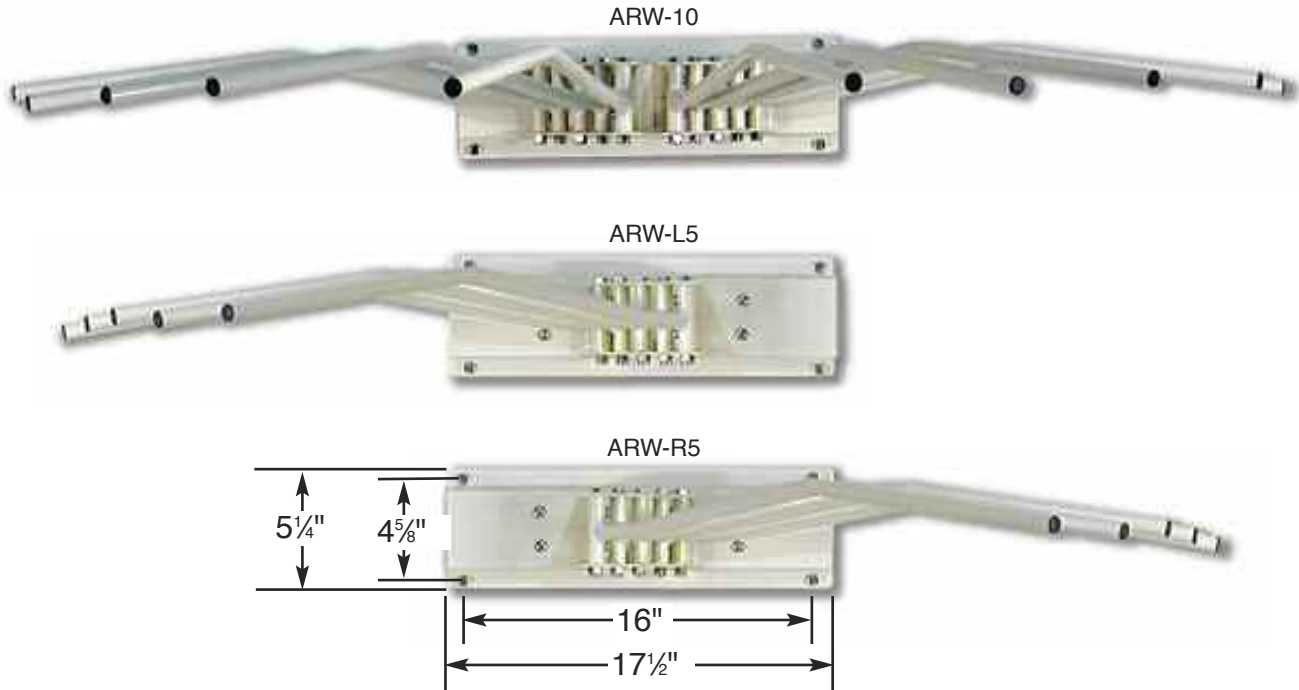

- (1) It is **IMPORTANT** that you find the **STRUCTURAL STUDS** in the wall before mounting this unit. Most walls are built with studs on either 16" or 24" centers. These units have 16" mounting centers; if your wall studs are on 24" centers, then you will need to order the #AGA-24 Adapter Plate.
- (2) The maximum space required for the swing-arc of the pivoting arm are:
 - ARW-10: 57 5/8" side-to-side and 29 1/2" front-to-back;
 - ARW-L5 & ARW-R5: 52 3/8" side-to-side and 29 1/2" front-to-back.

The excessive weight of lead aprons demands that you follow this procedure to insure proper function and utilization of the unit and also preserve the quality of your wall.