



USE & CARE INSTRUCTIONS FOR PROTECTIVE APPAREL

Thank you for choosing Bar-Ray for your new protective apparel. To ensure the longevity of your new garment, please follow these simple guidelines:

- Upon receipt of your garment, the Quality Control Officer should radiograph it to ensure that no damage occurred during shipping.
- When not in use your garment should be stored properly.
- Remove stains immediately when they occur. If this is not possible, clean with warm water and a non-bleach dishwashing soap. Do not use harsh or abrasive cleansers. Some upholstery cleaners that remove protein stains may be used to clean blood.

DO'S

- radiograph the apron on a regular basis.
- clean it regularly.
- store it properly.

DON'T'S

- sit in/on your apron.
- expose it to extreme temperatures.
- lean against sharp edges or pointed objects.
- store the apron over the back of a chair
- fold your apron

WARRANTY

One (1) year from date of shipment.

DISPOSAL

When ordering a new *Bar-Ray* protective apron, we offer disposal of your old *Bar-Ray* aprons on a one-for-one basis. Simply remove the outer cover, keeping the manufacturing tags intact, and return prepaid to the address shown below.

QUESTIONS

Please contact your LOCAL DEALER or sales representative.

Bar-Ray Products, Inc.
95 Monarch St.
Littlestown, PA 17340
Ph. 717-359-9100 Fax 717-359-9109